



Get into the Rhythm
of Regularity



866-359-7532
vibrantgastro.com



Clinically Proven to
Provide Predictable Relief⁴



According to leading gastroenterologist, Dr. Satish Rao, MD, some patients with constipation need to “**restore the natural biorhythm** of their gut. That is how Vibrant likely restores bowel rhythm.”

Have Constipation
Symptoms That
Won't Quit?



*Data on file at Vibrant Gastro.

1. Harris LA, Horn J, Kissous-Hunt M, et al. The better understanding and recognition of the disconnects, experiences, and needs of patients with chronic idiopathic constipation (BURDEN=CIC) study: Results of an online questionnaire. *Adv Ther* 2017;34:2661–2673. 2. Ron Y, Halpern Z, Safadi R, et al. Safety and efficacy of the vibrating capsule, an innovative nonpharmacological treatment modality for chronic constipation. *Neurogastroenterol Motil* 2015;27:99–104. 3. Rao SSC, Lembo A, Chey WD, et al. Effects of the vibrating capsule on colonic circadian rhythm and bowel symptoms in chronic idiopathic constipation. *Neurogastroenterol Motil* 2020;32:e13890. 4. Rao SSC, Quigley EMM, Chey WD, et al. Randomized placebo-controlled phase 3 trial of vibrating capsule for chronic constipation. *Gastroenterology* 2023;164:1202-1210.

137LAB Rev 01 – December 2023
© 2023 All rights reserved to Vibrant Gastro, Inc

Rely on Regularity

Effectively relieve constipation without:

- ✗ Unpredictable, untimely diarrhea
- ✗ Frequent straining and discomfort

- ✗ Away-from-home worries

Fight back with powerful
drug-free treatment.

Please see the Instructions For Use (IFU)
for full product and safety information.

Vibrant is approved for the treatment of chronic idiopathic constipation in adults who have not experienced relief of their bowel symptoms by using laxative therapies at the recommended dosage for at least one month.



Take Control

Get the ***predictable*** relief you need from chronic idiopathic constipation (CIC)—***without medication***.

60% of patients using branded prescription medications are dissatisfied.¹ Most find treatment ineffective, with unpleasant side effects. Life is unpredictable enough. You need to rely on regularity.



Get in Sync

Vibrant provides a unique mechanism of action likely to **restore the colon's natural rhythm**.

When swallowed, the innovative Vibrant capsule becomes activated and delivers micro-vibrations to stimulate your colon. The colon contracts reflexively, synchronizing with the body's biological rhythm to expel its contents.^{2,3}

Monitor Progress

During treatment, you and your healthcare provider can monitor bowel regularity through the **Vibrant Companion smartphone application**, designed to provide support and a better treatment experience. The app also makes it easy to remember when to take your capsules.^{2,3}

FDA-Cleared

The FDA-cleared Vibrant Treatment System Starter Kit includes:

- Activation Pod
- 20 Vibrant Capsules (1-month supply)
- AC/DC USB Adapter
- USB Cable



Never forget to take the capsule with **automatic app reminders**



Track your progress with an e-diary



Assist your physician in making data-driven decisions based on your sharing treatment progress reports



Get **interesting content** and relevant health tips



3 Easy Steps



1. ACTIVATE the capsule by placing it in the included pod. (Indicator will light green when capsule is ready.)



2. SWALLOW the capsule with about 8 ounces of water.



3. MONITOR relief on the Vibrant Companion app.

Ask Your Healthcare Practitioner

Vibrant is available through prescription only. It is conveniently delivered to your home via Carepoint™, our specialty pharmacy partner. As a recent innovation, your healthcare provider still may be unfamiliar with Vibrant.

Visit vibrantgastro.com/get-vibrant for a form to complete and share with your provider.

A Drug-Free Approach

A different kind of constipation relief, designed to make a difference. The Vibrant System is drug-free, provides for more frequent and predictable complete, spontaneous bowel movements, and is easy to use.

The capsules are intended to be orally ingested up to 5 times per week, as directed by a physician. The capsule should be taken shortly before going to sleep on 5 out of 7 days of the week according to the following schedule: 3 days on, 1 day off, 2 days on, 1 day off.

Please see the Instructions For Use (IFU) for full product and safety information.



Scan the QR code to learn more now.

866-359-7532
service@vibrantgastro.com

