


## The Pod

The pod is used to activate the Vibrant Capsule before swallowing the Capsule.

1. Designated Groove for Capsule
2. Light ring indicator
3. Pod lid

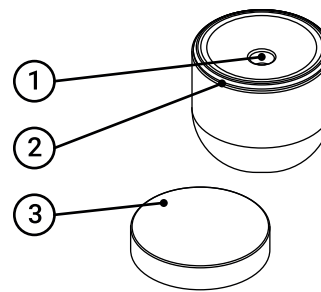

The Capsule should not cause any interference during travel activities such as flying and going through airport security.

## Who should not Use the Vibrant System?

You should **not** use the Vibrant System if you:

- Have a history of complicated/obstructive diverticular disease (small sacs in the colon)
- Have a history of, or have suspected, obstruction in your small intestine or colon
- Currently have significant gastroparesis (stomach is unable to empty itself)
- Have a history of significant gastrointestinal disorder, including any form of inflammatory bowel disease or gastrointestinal malignancy (celiac disease is okay if you have been treated and no longer experience signs or symptoms), and/or anal fissures and fistulas
- Have a history of Zenker's diverticulum (pouches in the lower part of the throat), dysphagia (trouble swallowing), esophageal stricture (abnormal tightening or narrowing in the throat), eosinophilic esophagitis (allergic condition in the throat) or achalasia (nerve damage in the throat that prevents swallowing)
- Are pregnant or lactating (breastfeeding)

## Important Safety Information

### Warnings:

⚠ The Vibrant Capsule is magnetic resonance (MR) unsafe. The device has not been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. Undergoing an MRI (magnetic resonance imaging) while the Capsule is inside your body may result in injury, including serious damage to your intestinal tract or abdominal cavity.

- If you require an MRI, instruct your physician to verify that no Capsules are currently in your body via abdominal X-ray before undergoing an MRI examination.

⚠ After ingesting a Vibrant Capsule and until it has left the body, you should not be near any source of powerful electromagnetic fields such as one created near an MRI device.

⚠ The Capsule should be kept away from implants such as pacemakers, defibrillators, nerve stimulators, and other devices that could be affected by proximity to a DC (direct current) magnetic field.

⚠ If you have reasonable doubt concerning the integrity of a Vibrant Capsule, it should not be used and should be discarded.

⚠ If after ingesting a Vibrant Capsule you will experience one of the following symptoms, contact a physician immediately and stop taking additional Capsules until instructed by your physician:

- Black, bloody or tarry stools;
- Abnormal abdominal pain, including new onset of abdominal pain, change in the character of a previously existing abdominal pain, or persistent abdominal pain;
- Nausea or vomiting; or
- Believe a Capsule has not been expelled from the body, after two weeks from the time of ingestion.

⚠ Under no circumstances should a Vibrant Capsule be swallowed without prior activation in the Pod.

⚠ Vibrant Capsules must be stored in a safe place, out of the reach of children and/or infants.

- If a child has accidentally swallowed an unused Vibrant Capsule, the child should be brought immediately to a hospital.