

# Storage and Maintenance Instructions








## Pod

- Keep the Pod in a dry place away from direct sunlight (not in the kitchen or in the bathroom).
- Thoroughly clean the Pod prior to first use and after each use. The Pod should be cleaned using a clean cloth and 70% isopropyl alcohol, and the lid should be closed after cleaning and between uses.
- In the case of electromagnetic disturbance, reset the Pod by unplugging the AC/DC adaptor from the wall power outlet and plugging it back in after a few seconds.

## Capsules

- The Vibrant Capsules should be stored in a dry, cool place (below 25°C/77°F), away from magnetic sources and away from direct sunlight (not in the kitchen or in the bathroom).
- Keep the Vibrant Capsules in the sealed blister pack until use.
- Even if stored in their original containers and according to recommendations, the Vibrant Capsules can be kept for a limited period only. Please note the expiration date of the Capsules.

# Symbols

	Federal (U.S.) law restricts this device to sale by or on the order of a physician.
	The Capsule component is intended for single use
	Keep dry
	Protected from touch by fingers and objects greater than 12 mm
	Type BF applied part
	For additional information on how to use the Vibrant System, please call: 866- 359-7532 (toll free)
	For additional information regarding the Vibrant System, you may visit our website at <a href="http://www.vibrantgastro.com">www.vibrantgastro.com</a>

US Office Address: 9600 Great Hills Trail, Suite 150W, Austin, Texas, 78759  
Manufactured by Vibrant LTD, Hakochav Building, 2 Hacarmel St., Yokneam 2069206, Israel